

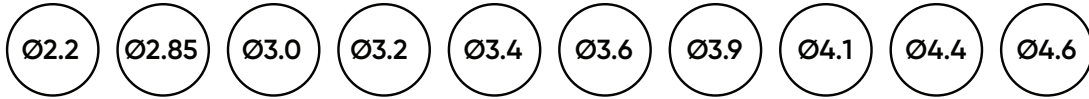
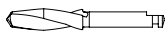


Drilling Protocol for Straight, Tapered and Edge ProActive Implants

Twist Drills

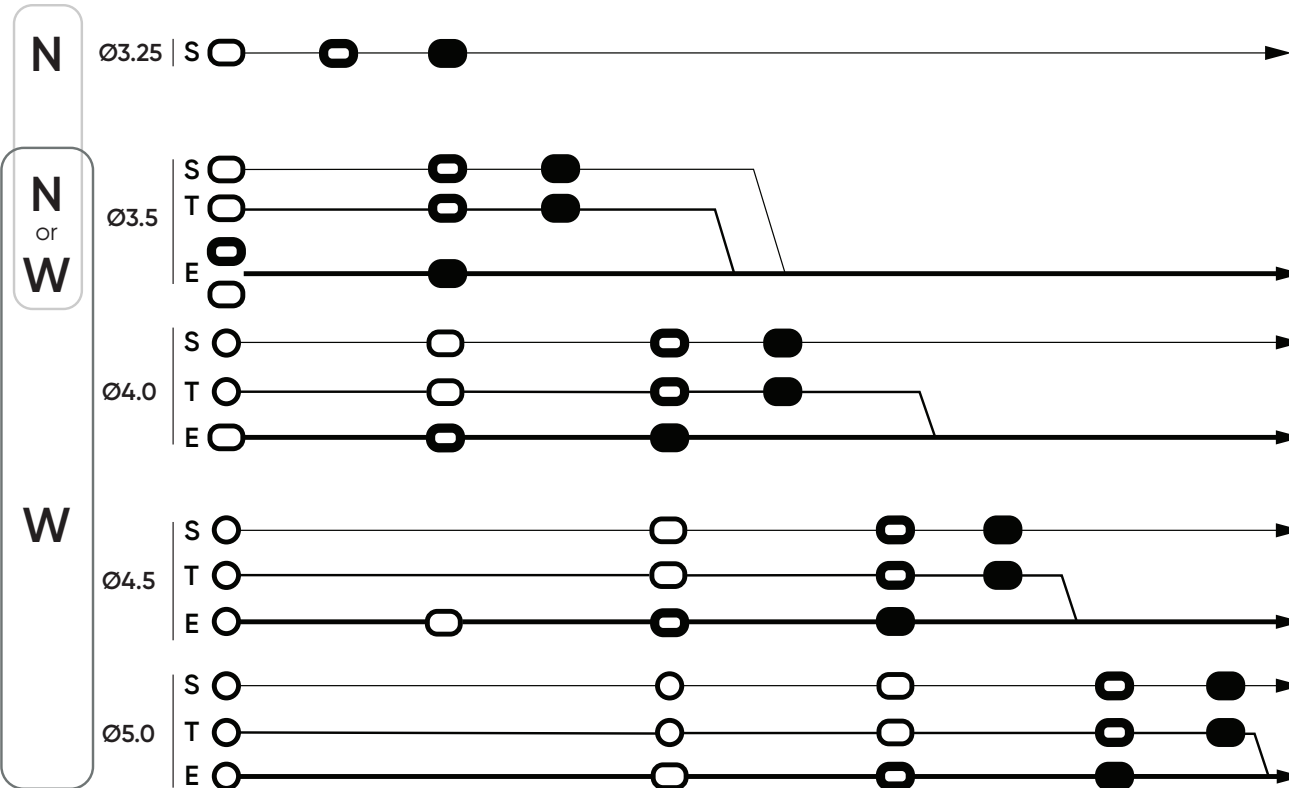

Countersinks

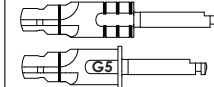

Ø3.25 S
N

Ø3.5 S/T/E
N - W

Ø4.0 S
W

Ø4.0 T/E
W


Ø4.5 S
W

Ø4.5 T/E
W

Ø5.0 S
W

Ø5.0 T/E
W

Example of marking on Drills

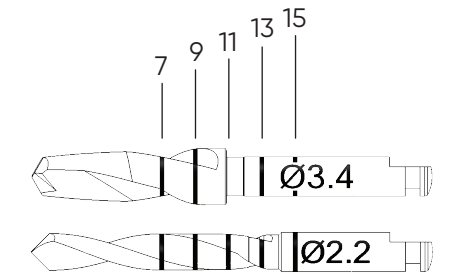

Drilling Steps / Bone Quality

- Mandatory drill step
- Final drill for Soft bone
- Final drill for Regular bone
- Final drill for Dense bone

		Sleeve Position		
		G3	G5	G7
Implant Length	7 mm		I	II
	9 mm	I	II	III
	11 mm	II	III	IV
	13 mm	III	IV	V
	15 mm	IV	V	