

The shape  
of things  
to come



**neoss®**

Esthetic Healing Abutments  
with ScanPeg

Intelligent Simplicity

## Esthetic Healing Abutments with ScanPeg

---

**A complete range of anatomically-shaped PEEK healing abutments with a dedicated ScanPeg, for simplified and accurate intra-oral scanning.**



Easier handling – a faster, improved procedure

- Minimized component exchange
- Streamlined intra-oral scanning
- Less chair time, fewer treatment steps

Undisturbed healing and more patient-friendly

- Biological seal is maintained, tissue level is preserved
- Less invasive for increased patient comfort

Anatomically-shaped to create excellent esthetics

- Shapes soft tissue during healing
- Perfectly matched with Neoss Esthetiline final abutments and Neoss customized abutments


A smarter, more efficient process for you

A more comfortable experience for your patients

---

## 01 Healing abutment placement

The Esthetic Healing Abutment is placed during first or second stage surgery and then left until the final abutment is placed.

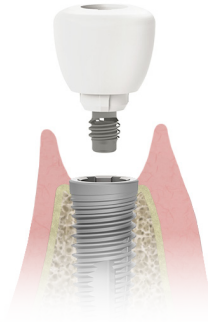


## 02 ScanPeg placement and scanning

An accurate intra-oral scan is carried out at healing abutment level utilizing the ScanPeg.


The ScanPeg comes co-packed with the Esthetic Healing Abutment and is easy to both attach and remove.

The intra-oral digital impression can be taken whenever you choose, using your existing scanner.

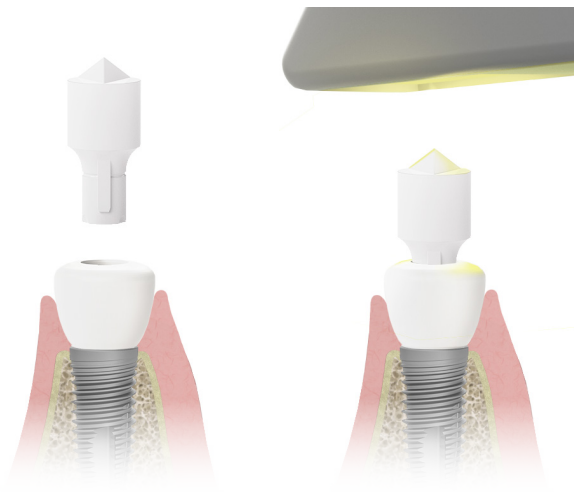

## 03 Placement of the final restoration

Flexibility in when you take the digital impression allows design work to continue during the healing phase. The healing abutment is not removed until the final restoration is placed, eliminating one complete surgical intervention.

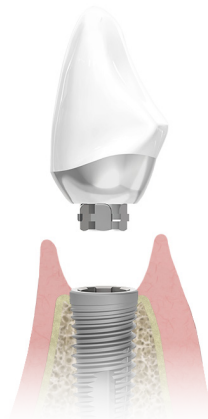

This new range of Esthetic Healing Abutments with intra-oral scanning capabilities is yet another example of our unique 'Intelligent Simplicity' in action. Innovative and effective solutions which put you and your patient first.

# Assortment

## Art. No. Description

31360	Esthetic Healing Abutment with ScanPeg – Wide Incisor
31361	Esthetic Healing Abutment with ScanPeg – Narrow Incisor
31362	Esthetic Healing Abutment with ScanPeg – Canine
31363	Esthetic Healing Abutment with ScanPeg – Pre-molar
31364	Esthetic Healing Abutment with ScanPeg – Molar

31365	ScanPeg – 5 pcs
-------	-----------------

Part of the Neoss Esthetiline range

Contact your local representative for information about availability on your market and list of compatible intra-oral scanners and design software.

# Contact details

## Australia

Neoss Australia Pty. Ltd  
PO Box 404  
New Farm  
QLD 4005  
T +61 7 3216 0165  
F +61 7 3216 0135  
E info.au@neoss.com

## Austria

Neoss GmbH  
T +49 221 55405-322  
F +49 221 55405-522  
E info@neoss.de

## China

Shanghai LegenDental Medical Technology Co. Ltd  
4th floor, Block 6  
1525 Mingqiang Road  
Songjiang District  
Shanghai 201612  
P. R. China  
T +86-21-67657130  
F +86-21-67657131  
E info@legendental.cn

## Croatia

Medical Intertrade d.o.o  
10431 Sveta Nedelja  
HRVATSKA  
T +385 1 33 74 020  
F +385 1 33 36 035  
E dental@medical-intertrade.hr

## Denmark

Neoss AB  
T +46 (0)31 88 12 80  
E info@neoss.se

## Germany

Neoss GmbH  
Im MediaPark 8  
DE-50670 Köln  
T +49 221 55405-322  
F +49 221 55405-522  
E info@neoss.de

## Italy

Neoss Italia S.r.l.  
Via Marco Antonio Colonna, 42  
I-20149 Milano  
T +39 02 92952 1  
F +39 02 92952 250  
E italia@neoss.info

## Japan

Neoss Japan Co.  
Y101 Building 5F  
No. 1-4 Kanda-Sudacho Chiyodaku  
Tokyo 101-0041  
T +81 3 5289 3511  
F +81 3 5289 3512

## Middle East & Africa

Neoss Limited  
30th Floor Office 19, Reef Tower JLT Cluster (O)  
Dubai  
P.O. Box 5003315  
United Arab Emirates  
T +971 4 448 75 77  
F +971 4 448 71 12  
E info.me@neoss.com

## Netherlands

Rotor, Medical Supplies  
Economiestraat 16  
NL-6135 KV Sittard  
T +31-46-481 12 03  
F +31-46-481 12 04  
E mail@rotor-medical.com

## New Zealand

Neoss Australia Pty. Ltd  
T +61 7 3216 0165  
F +61 7 3216 0135  
E info.au@neoss.com

## Norway

Neoss AB  
T +46 (0)31 88 12 80  
E info@neoss.se

## Republic of Ireland

Neoss Ltd  
T +44 1423 817-733  
F +44 1423 817-744  
E info@neoss.com

## Spain

Formación, Gestión Y Rendimiento, S.L  
C. Londres 54 E2B  
ES-08036 Barcelona  
T +34 931 408 770  
E info@fgrdental.com

## Sweden

Neoss AB  
Arvid Wallgrens backe 20  
SE-413 46 Göteborg  
T +46 (0)31 88 12 80  
E info@neoss.se

## Switzerland

Flexident AG  
Schützenmatte B 11  
Postfach 453  
CH-6362 Stansstad  
T +41413104020  
F +41413104025  
E info@flexident.ch

## Thailand

ACTEON (Thailand) Limited  
23/45 Sorachai Building,  
16th Floor Sukumvit 63 Road  
Klongton Nua – Wattana  
BANGKOK 10110  
T +66 2 714 3295  
F +66 2 714 3296  
E info-thailand@acteongroup.com

## Turkey

Neoss Sağlık Hizmetleri Anonim Şirketi  
Küçükbakkalköy mh. Kocasinan cad. No:4  
1 Atasehir  
34750 İstanbul  
Turkey  
T +9 0216 504 3 444  
E info@neosshsi.com

## United Kingdom

Neoss Ltd  
Windsor House  
Cornwall Road  
Harrogate  
HG1 2PW  
T +44 1423 817-733  
F +44 1423 817-744  
E info@neoss.com

## United States

Neoss Inc.  
21860 Burbank Blvd.,  
Ste. 190 (North Entrance)  
Woodland Hills, CA 91367  
T +1 866 626-3677  
F +1 818 432-2640  
E contact.usa@neoss.com