

Assistant Handbook








2. Assistant Handbook

Section	Page
2.1 Surgical Assistant Handbook	2:3
2.1.1 General Features	2:3
2.1.2 Treatment Options	2:4
2.2 Surgical Procedure and Drilling Protocol	2:4
2.2.1 Surgery Set-up	2:4
2.2.2 Surgical Procedure	2:9
2.2.3 Drilling Protocols	2:10
2.2.4 Surgical Drills	2:13
2.3 Restorative Assistant Handbook	2:17
2.3.1 Prosthetic Tray and Instrument Kit	2:17
2.4 Cleaning, Disinfection, Sterilization, Storage and Lifetime	2:18
2.5 Oral Hygiene and Patient Care	2:18
2.6 General Packaging Symbols	2:19

2.1 Surgical Assistant Handbook

The Neoss® Implant System is available in 7 implant diameters Ø3.5, Ø4.0, Ø4.5, Ø5.0, Ø5.5, Ø6.0 and Ø6.5, all with the prosthetic Standard Platform (SP), and in addition there is a narrow Neoss Ø3.25 mm implant with the prosthetic Narrow Platform (NP). The implants are generally available in lengths from 7–17 mm, please refer to product catalog for detailed information about available implant types, diameters and lengths. The packaging for Neoss implants and instruments used for a specific implant diameter (countersinks and screwtaps) have the following color coding:

- | | | |
|--|---|--|
|  Ø3.25 mm <i>Royal Blue</i> |  Ø4.0 mm <i>Yellow</i> |  Ø5.0 mm <i>Peach</i> |
|  Ø3.5 mm <i>Green</i> |  Ø4.5 mm <i>Blue</i> |  Ø5.5 mm <i>Lilac</i> |

The Neoss implants have a universal self-cutting design making them suitable for all bone qualities.

The implant is 'picked up' from a sterile glass vial with an Implant Inserter. The surgical drills are for single use and delivered in sterile condition for immediate use. There is only one screwdriver connection in the standard assortment, the Neo screwdriver, and this is used for all components including cover screws, healing abutment screws, and final abutment screws.

Neoss implants kits includes a cover screw, two healing abutments (only 5 mm with Ø3.25 mm implant) and a healing screw. This complete delivery method enables the clinician to undertake either one or two stage surgery at time of placement without the need to have pre-ordered individual components. There are also two stickers provided in the implant kit to assist in recording information on the patient's chart. The ProActive Edge implants are also available with a cover screw only.

The following information is a guide as requirements may vary on an individual basis.

2.1.1 General Features

The Neoss Implant System provides a simple, easy to use means of anchorage for a single crown, bridge or denture thereby satisfying a wide range of aesthetic and functional requirements. Simple implant installation and flexibility in prosthetic solutions provides optimal aesthetic restorations for a wide range of clinical situations. This handbook serve as a clinical reference for surgical and restorative assistant procedures.

The Neoss Implant System

The Neoss Implants are based on extensive research and development, the outcome of which is a state-of-the-art system, rationalized by design. The implants have patented design and geometry which imparts specific features and benefits to the system.

Neoss implants may be used as a one or two-stage implant and are manufactured from Commercially Pure Titanium Grade IV with a surface that has been subjected to a multistage blasting, etching, cleaning and chemical treatment. The system fulfills all clinical indications with a compact and rational range of implant components and instruments.

The Neoss implant to abutment connection

Unique to the Neoss Implant System is the ONE prosthetic platform, across three implant ranges. The same prosthetic components fit every standard implant. All standard Neoss implants, Ø3.5 and larger, have the same standard platform (SP) with the implant to abutment connection design called NeoLoc®. For Ø3.25 mm implants the implant connection has a smaller narrow platform (NP).

2.1.2 Treatment Options

The Neoss implants may be placed using a Single/One Stage Surgical Protocol (which may involve immediate loading/function) or a Two Stage surgical protocol.

Either surgical protocol may be used to construct a single tooth, bridge or overdenture. Factors which may influence the choice of one protocol over the other are detailed in the Neoss Implant System Surgical Handbook.

- Single/One Stage Surgery – this procedure involves placing a healing abutment, a provisional abutment or prosthesis at time of implant placement.
- Two Stage Surgery – this procedure involves placing a cover screw at the time of implant placement, then after a designated healing time a second surgical procedure to uncover the implant and place a healing/provisional or other form of abutment.

Prior to the actual procedure, treatment objectives and goals should have been discussed with the patient and careful planning in relation to the number and diameter of implants have been determined.

2.2 Surgical Procedure and Drilling Protocol _____

2.2.1 Surgery Set-up

Either an operating theatre or a well prepared dental surgery may be used for the procedure.

Suggested surgical items/instruments – GENERAL:

- caps, gloves, gowns and masks
- drapes for patient
- additional drapes for bench tops, stands etc.
- suction equipment
- irrigation equipment
- antiseptic solution/clamp and swabs for patient preparation
- surgical instruments: scalpels, mirror, bowl, cheek retractors, elevators, scissors – dissecting/suture, forceps, artery forceps
- gauze, gauze swabs etc.
- tubing covers
- anaesthetic/syringe
- drilling equipment, handpiece and motor

Suggested surgical items/instruments – NEOSS SYSTEM (please refer to flowchart on the following pages):

- drill kit, optional drills, countersink, screw tap
- implants
- pre-sterilized surgical and prosthetic tray
- Neoss System surgical instruments: drill extender, inserters 17/22/32 mm (NP 24/32 mm), Wrench Adapter, Neo screwdrivers 22/32 mm, 15 mm manual Neo screwdriver, manual handle, ratchet, direction depth gauges, Depth Gauge Probe, Insertion Handle – Large
- Neoss ProActive Implant System Box (fits the surgical and prosthetic trays – used for sterilizing and storing instruments)

Handling of hazardous material according to established procedures at the hospital/clinic.

Implants
ProActive Edge

Drilling Sequence
Recommended (Regular)

Optional

Ø3.5



- 9 mm #21301
- 11 mm #21302
- 13 mm #21303
- 15 mm #21304
- 17 mm #21421



Ø2.2 S



Ø3.5 E
Countersink



Ø3.0 T

Ø4.0



- 9 mm #21305
- 11 mm #21306
- 13 mm #21307
- 15 mm #21308
- 17 mm #21427



Ø2.2 S



Ø3.0 T



Ø4.0 E
Countersink



Ø3.4 T

Ø4.5



- 9 mm #21309
- 11 mm #21310
- 13 mm #21311
- 15 mm #21312
- 17 mm #21433



Ø2.2 S



Ø3.4 T



Ø4.5 E
Countersink



Ø3.0 T



Ø3.9 T

Ø5.0



- 9 mm #21313
- 11 mm #21314
- 13 mm #21315
- 15 mm #21316



Ø2.2 S



Ø3.9 T



Ø5.0 E
Countersink



Ø3.4 T



Ø4.4 T

Implants

ProActive Tapered

Ø3.5



9 mm #21221
11 mm #21222
13 mm #21223
15 mm #21224



Ø2.2 S Ø3.0 T

Optional



Ø3.2 S
Ø3.5 T Countersink
Ø3.5 Screw Tap


Ø4.0



9 mm #21227
11 mm #21228
13 mm #21229
15 mm #21230



Ø2.2 S Ø3.4 T

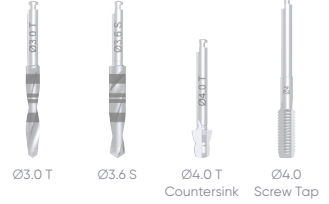

Ø3.0 T Ø3.6 S
Ø4.0 T Countersink
Ø4.0 Screw Tap

Ø4.5



9 mm #21233
11 mm #21234
13 mm #21235
15 mm #21236



Ø2.2 S Ø3.9 T



Ø3.4 T Ø4.1 S
Ø4.5 T Countersink
Ø4.5 Screw Tap

Ø5.0



9 mm #21239
11 mm #21240
13 mm #21241
15 mm #21242



Ø2.2 S Ø3.4 T Ø4.4 T



Ø3.9 T Ø4.6 S
Ø5.0 T Countersink
Ø5.0 Screw Tap

Ø5.5



9 mm #21245
11 mm #21246
13 mm #21247



Ø2.2 S Ø3.4 T Ø4.9 T



Ø4.4 T Ø5.1 S
Ø5.5 T Countersink
Ø5.5 Screw Tap

ProActive Ø6.0



7 mm #21252
9 mm #21250
11 mm #21251



Ø2.2 S Ø3.4 T Ø4.9 T Ø5.5 T



Ø6.0
Countersink

Implants

ProActive Straight

Ø3.25



9 mm #21176
 11 mm #21177
 13 mm #21178
 15 mm #21179

Drilling Sequence Recommended (Regular)



Optional



Ø3.5



7 mm #21181
 9 mm #21182
 11 mm #21183
 13 mm #21184
 15 mm #21185
 17 mm #21186



Ø4.0



7 mm #21187
 9 mm #21188
 11 mm #21189
 13 mm #21190
 15 mm #21191
 17 mm #21192



Ø4.5



7 mm #21193
 9 mm #21194
 11 mm #21195
 13 mm #21196
 15 mm #21197
 17 mm #21198



Ø5.0




7 mm #21199
 9 mm #21200
 11 mm #21201
 13 mm #21202
 15 mm #21203



Ø5.5



7 mm #21205
 9 mm #21206
 11 mm #21207
 13 mm #21208

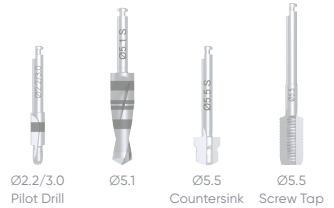

Instruments

Implant Inserters and Wrench Adapter



Neo & iGO Screwdrivers and Manual Handle



Insertion Handle – Large
 #51191



Ratchet – Torque Driver
 #51121



Bone Remover Cover Screw
 SP #41240
 NP #41241

Bone Profiler
 SP #41237
 SP #41238
 NP #41239



Drill Extender
 #41120


Direction Depth Gauge
 #51125 pkt of 4



Depth Gauge Probe
 #51190


Implant Kit

(not all implants sold in kits)

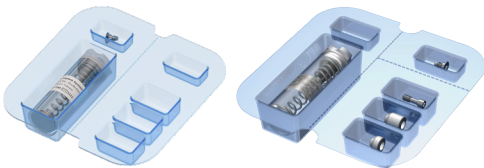

Contents of Kit:
 Implant, Cover Screw, Healing Abutment with Screw

Surgical and Prosthetic Trays

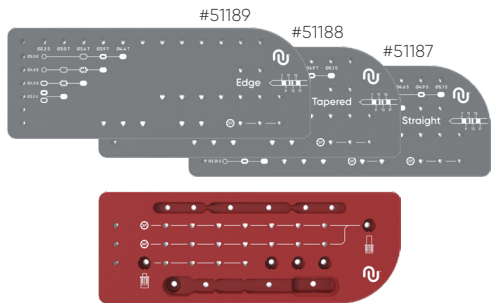

Titanium Healing Abutments

Esthetic Healing Abutments



2.2.2 Surgical Procedure

The surgical procedure may entail a range of procedures including minimally invasive surgery and raising a full thickness flap and exposing the bone in the proposed site. A series of increasing diameter drills are used to enlarge the osteotomy for implant placement – this may involve the use of countersinks and screw taps depending on individual preference and/or the quality of bone.

- If the procedure is to be carried out in a hospital environment then the preparation of the theatre and surgical staff should conform to the established protocols of each individual hospital.
- It is desirable to have both a sterile and non-sterile assistant throughout the procedure. Ensure sterile handling during preparation and surgery.
- All bone preparation drilling is carried out under profuse irrigation using either saline or sterile water to avoid overheating of the bone.
- If a surgical guide/stent is to be used for implant placement then follow the manufacturer's recommendation for the sterilization procedure.
- The drilling sequence for bone preparation is outlined in the Neoss System Drilling Protocols (following pages) however individual preferences or bone quality may require a deviation from these protocols. It is therefore recommended that additional/optional components only be opened when indicated by the surgeon.

Note: Please refer to the Neoss Implant System Surgical Handbook for detailed information in relation to:

- *Machine implant insertion*
- *Manual implant insertion*
- *Single stage surgical procedure*
- *Two stage surgical procedure*
- *Post operative care*

2.2.3 Drilling Protocols

ProActive Edge Implants

		BONE QUALITY					
		IV & III Soft		II Regular		I Dense	
Begin with starter drill: - lance drill - round bur		Ø2.2 S	Ø3.0 T	Ø3.4 T	Ø3.9 T	Ø4.4 T	Countersink
Ø5.0 mm							Ø5.0 E
Ø4.5 mm							Ø4.5 E
Ø4.0 mm							Ø4.0 E
Ø3.5 mm							Ø3.5 E
Drill Stop							

Guidelines


1. Start by identifying the diameter of the implant. The first drill is always the Ø2.2 mm twist drill.
2. Move to the next marking and prepare the site with the drill corresponding to that marking. The dashed drill step can be skipped if it is not the final drill step for the selected bone quality.
3. Keep moving right until the final preparation is performed at the chosen bone quality marking.
4. The use of countersink is recommended, but is not required in situations where underpreparation of the cortical bone is desired.

Note: In presence of dense bone, additional care is taken during insertion. The thread cutting and forming design of the implant acts as a screw tap. If high insertion torque occurs, back the implant out half to one turn before continuing.

The guiding portion of the Edge countersink is designed to match the drill for standard bone. If a narrower osteotomy for soft bone needs to be countersunk, it may be necessary to first widen the cortical portion of the osteotomy with the drill for standard bone so that the countersink can seat correctly.

ProActive Tapered Implants

BONE QUALITY |  Soft IV & III |  Regular II |  Dense I

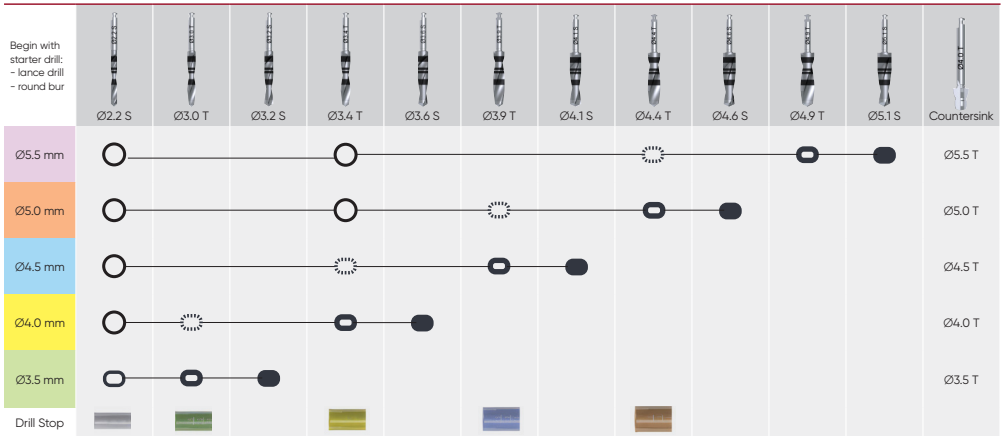

Guidelines

1. Start by identifying the diameter of the implant. The first drill is always the Ø2.2 mm twist drill.
2. Move to the next marking and prepare the site with the drill corresponding to that marking. The dashed drill step can be skipped if it is not the final drill step for the selected bone quality.
3. Keep moving right until the final preparation is performed at the chosen bone quality marking.
4. The use of countersink is recommended, but is not required in situations where underpreparation of the cortical bone is desired.


Note: In presence of dense bone, additional care is taken during insertion. The thread cutting and forming design of the implant acts as a screw tap. If high insertion torque occurs, back the implant out half to one turn before continuing.

ProActive Straight implants

BONE QUALITY

○ Regular
||

● Dense
|

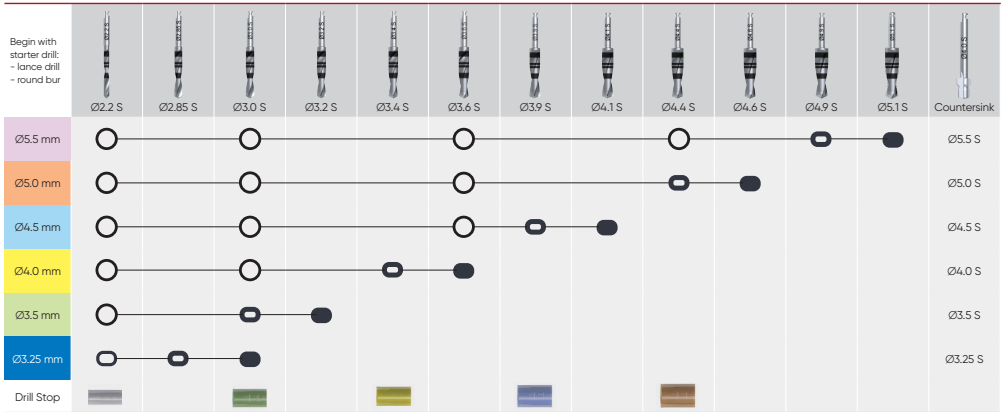

Guidelines

1. Start by identifying the diameter of the implant. The first drill is always the Ø2.2 mm twist drill.
2. Move to the next marking and prepare the site with the drill corresponding to that marking. The dashed drill step can be skipped if it is not the final drill step for the selected bone quality.
3. Keep moving right until the final preparation is performed at the chosen bone quality marking.
4. The use of countersink is recommended, but is not required in situations where underpreparation of the cortical bone is desired.

Note: In presence of dense bone, additional care is taken during insertion. The thread cutting and forming design of the implant acts as a screw tap. If high insertion torque occurs, back the implant out half to one turn before continuing.

ProActive Wide Ø6.0 & Sinus Ø6.5 implants






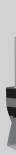





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







BONE QUALITY



Soft
IV & III



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Regular
II

Begin with starter drill: - lance drill - round bur							
	Ø2.2 S	Ø3.4 T	Ø4.9 T	Ø5.1 S	Ø5.5 T	Ø6.0 S	Countersink
Ø6.5 mm							Ø6.5
Ø6.0 mm							Ø6.0

Guidelines

1. Start by identifying the diameter of the implant. The first drill is always the Ø2.2 mm twist drill.
2. Move to the next marking and prepare the site with the drill corresponding to that marking. The dashed drill step can be skipped if it is not the final drill step for the selected bone quality.
3. Keep moving right until the final preparation is performed at the chosen bone quality marking.
4. The use of countersink is recommended, but is not required in situations where underpreparation of the cortical bone is desired.

2.2.4 Surgical Drills

The Neoss Implant System is available in 7 diameters Ø3.5, Ø4.0, Ø4.5, Ø5.0, Ø5.5, Ø6.0 and Ø6.5, all with the prosthetic Standard Platform (SP), and in addition there is a narrow Neoss Ø3.25 mm implant with the prosthetic Narrow Platform (NP). Neoss Implant System Drill Kits contain the recommended drills for the placement of Neoss Edge, Tapered and Straight implants. All for regular bone Drills, Countersinks and Screw Taps are available separately. Neoss offers drills for single use (single patient only) which are delivered in a sterile condition for immediate use. If the sterile barrier is broken the drills can be re-sterilized, described in section 2.4.

Neoss also offers drills for multiple use.

Art. No.	Items Included
41177	Drill Kit, Edge Implants Ø3.5–5.0
41192	Drill Kit, Tapered Implants Ø3.5–5.0
41193	Drill Kit, Straight Implants Ø3.5–5.0
51189	Neoss Surgical and Prosthetic Tray – ProActive® Edge
51188	Neoss Surgical and Prosthetic Tray – ProActive® Tapered
51187	Neoss Surgical and Prosthetic Tray – ProActive® Straight

Note: Specific lasermarkings on shafts for identification: E for Edge, T for Tapered, S for Straight.


Surgical and Prosthetic Trays

The Neoss Surgical and Prosthetic Trays are designed as two interlocking parts for surgery, instruments and layout. These can be used in combination or individually. Made of highly durable silicone they are easily cleaned and sterilized (100 cycles and up to 1 year).

The grey surgical part of the tray offers clear markings for drill selection and depth on one side and storage for instruments during sterilization on the other.


Note: The red part offers markings for instruments needed during installation of implants and abutments. The surgical and prosthetic trays are marked with 'Edge', 'Tapered' or 'Straight' respectively.

Note: It is possible to combine the drill set-up sections for ProActive Edge, ProActive Tapered and ProActive Straight implants.

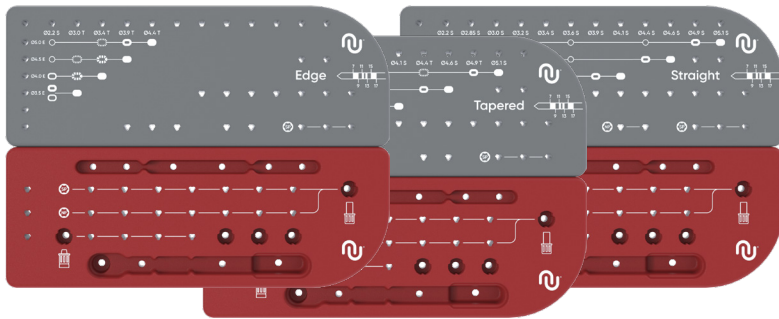


ProActive Edge Implants

Surgical and prosthetic setup

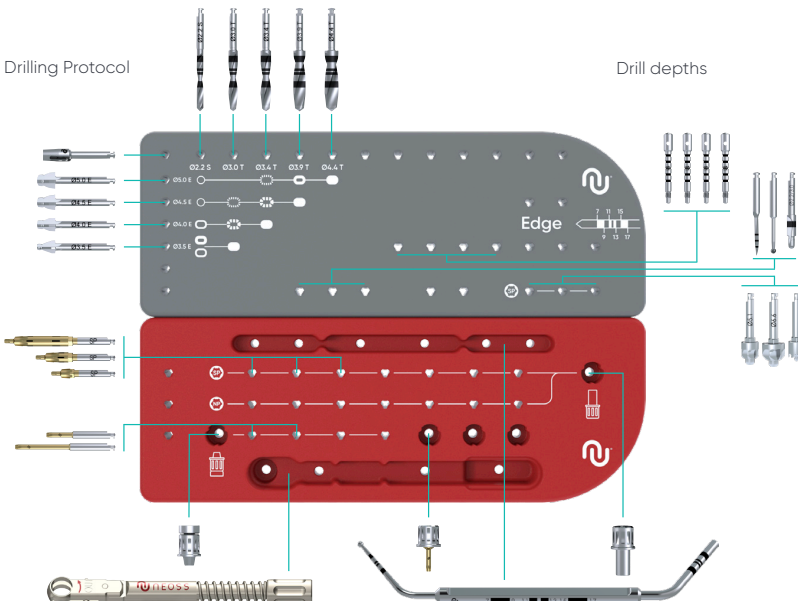


ProActive Tapered Implants

Surgical and prosthetic setup

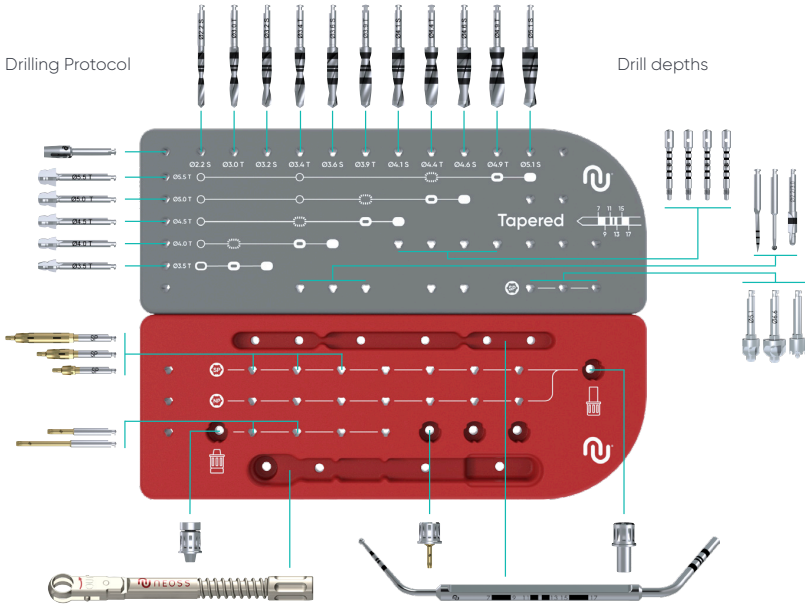

ProActive Straight Implants

Surgical and prosthetic setup

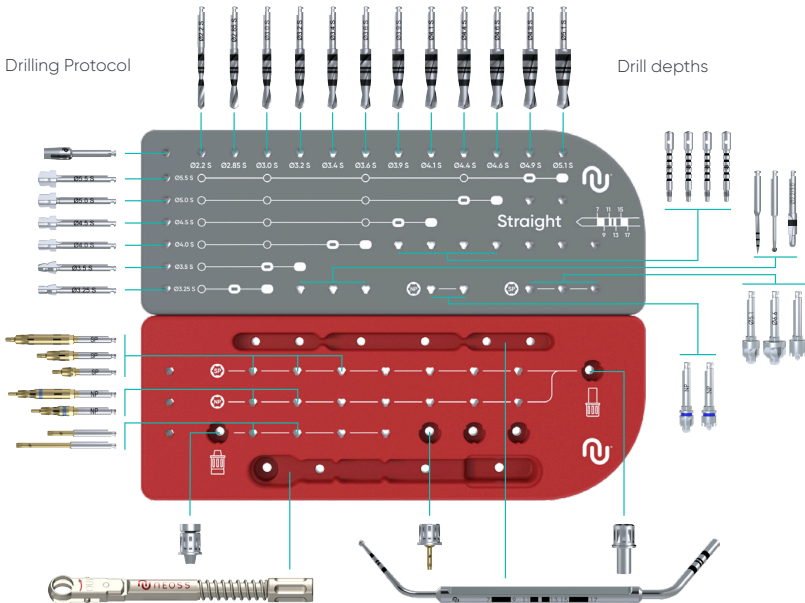


Neoss Drill Stops

Neoss drill stop solution satisfies all clinical needs and provides improved safety, control and efficiency. The Drill Stops enable precise depth control during preparation of implant sites for the placement of Neoss System implants. Neoss Drill Stops are compatible with Neoss drills with corresponding diameters including Neoss Tapered drills.

The assortment consists of a separate kit for implant lengths 7 – 15 mm. Each kit includes five Drill Stops of different diameters which correspond to final recommended drill diameters in regular bone. These are delivered sterile and are color coded: clear $\varnothing 2.2$, green $\varnothing 3.0$, yellow $\varnothing 3.4$, blue $\varnothing 3.9$ and peach $\varnothing 4.4$.



Drill stop marking

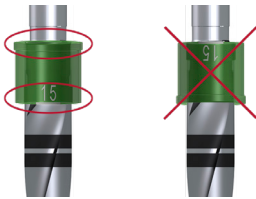

Correct mounting

Incorrect mounting

Clinical Procedure

The Drill Stop is mounted on the corresponding drill and secured by a light push. Ensure that the mounted Drill Stop is correctly chosen and seated to the right depth by checking the corresponding depth marking on the drill. After use, the drill stop is removed by a light pull and discarded. The Drill Stops are single use only.

Note: The drill stop must be mounted with the flange and marking directions as shown.

Note: Neoss Short Drills (7 – 13 mm) are NOT compatible with Neoss Drill Stops.

Contraindications

Neoss Drill Stops are not indicated in extraction sites as it may be difficult to accurately judge the depth of the stop.

In cases with uneven bone, the drill stops have to be removed for complete or partly submerged implant placement.

Example

Preparing an implant site for a 4 × 11 mm implant requires use of $\varnothing 2.2$, 3.0 and 3.4 mm drill stops from Neoss Drill Stop 11 mm.



Depth guide

2.3 Restorative Assistant Handbook

The Neoss Implant System is available in 7 diameters Ø3.5, Ø4.0, Ø4.5, Ø5.0, Ø5.5, Ø6.0 and Ø6.5, all with the prosthetic Standard Platform (SP), and in addition there is a narrow Neoss Ø3.25 mm implant with the prosthetic Narrow Platform (NP).

Neoss engaging abutments have deformation lugs which minimise rotational movements and secures a distinct seating.

There is only one screwdriver connection in the assortment, the Neo screwdriver, and this is used for all components including cover screws, healing abutment screws, and final abutment screws.

Neo Abutment Screw is a high performance screw which enables a high clamping force between the abutment and the implant.

Generally the patient will present to the restorative surgery with a healing abutment in place. In the majority of cases the impression will be taken at 'Implant Level', however some abutments allow for their preparation intraorally – similar to that of a natural tooth – in these cases a conventional crown and bridge impression protocol would be followed.

The Neoss System offers one universal Implant Level Impression Coping for both 'Open' and 'Closed' Tray impression techniques and one Impression Coping for 'Open Tray' impression only.

The Neoss Implant System offers patients a broad range of aesthetic and functional solutions.

These are available as cemented or screw-retained options, overdenture and CAD/CAM designed prostheses.

Note: Please refer to the information in this manual for procedures and information in relation to:

- Esthetiline Solution
- Provisional Abutments
- Impression Techniques
- NeoLink® – Gold/Titanium
- Single Unit and Multiple Unit Construction
- Titanium Prepable Abutments
- Zirconia Abutments
- Access Abutments
- Overdenture Solutions

2.3.1 Prosthetic Tray and Instrument Kit

The tray holds the Neoss ratchet, manual handle and Neo screwdrivers and includes spare slots for additional components. The lid is easy to remove, and the base design allows for easy access to instruments.

Made of a highly durable silicone and with no grommets, the tray is easily cleaned and sterilized (100 cycles and up to 1 year).



2.4 Cleaning, Disinfection, Sterilization, Storage and Lifetime

Regarding instructions for cleaning, disinfecting, sterilization and storage of Neoss Reusable Products and all prosthetic products for invasive use, please refer to Guideline for cleaning & maintenance Neoss reusable products (14077).

All prosthetic products and dental instruments that are delivered non-sterile must after removal of the protective transport packaging be cleaned and if required sterilized before use. This also applies for adjusted abutments coming from lab.

Storage

Sterilized bags are stored in dry environment at room temperature.

Lifetime of reusable products

Please refer to Guideline for cleaning & maintenance Neoss reusable products (14077) to find guidelines how to decide when a reusable product is outworn and needs to be exchanged.

2.5 Oral Hygiene and Patient Care

As with natural dentition, dental implants/prosthesis are susceptible to plaque build-up which may have a detrimental affect on the long term success of the prosthesis. It is therefore of vital importance that the patient is carefully instructed on the importance of regular check-ups and 'home care'. Following insertion of the final prosthesis the patient should be instructed in the routine for home care.










When instructing patients how to maintain their implant supported prosthesis it should be remembered that some patients may not have had natural teeth for some time. Therefore individualized and thorough instruction on 'how to clean' should be developed for each patient. This may include the recommendation of certain toothbrushes, mouth rinses, dental floss or interdental cleaning aids.

Titanium is a soft metal and therefore the use of abrasive toothpastes or instruments which may scratch the abutment should be avoided.

In addition to 'home care' it is recommended that the patient be checked regularly in the first 12 months after prosthesis insertion. The dentist would include in the check-up the stability of the prosthesis, the occlusion, surrounding soft tissues and the patient's ability to maintain a high level of 'at home' oral hygiene.

2.6 General Packaging Symbols _____

*Single use devices should not be reused due to risks of product contamination, patient/user infection and/or failure of the device to perform as intended.

USE BY/EXPIRY DATE 	CATALOGUE NUMBER REF	LOT/BATCH NUMBER LOT	DO NOT RE-USE (Single use only)* 	TEMPERATURE LIMIT 
KEEP AWAY FROM SUNLIGHT 	MANUFACTURER 	DATE OF MANUFACTURE 	STERILIZED USING ETHYLENE OXIDE STERILE EO	STERILE BY IRRADIATION (Contents of inner package sterile) STERILE R
NON-STERILE 	DO NOT USE IF PACKAGE IS DAMAGED 	MEDICAL DEVICE MD	CONSULT INSTRUCTIONS FOR USE (Also available on www.neoss.com/IFU) 	CAUTION: Federal (USA) law restricts the sale of this device to or on the order of a licensed physician or dentist Rx only